

New Feature

Clearing Single Viewport

Overview

Adds the ability to clear contents from a single viewport instead of clearing all viewports at once. This is a sub-feature of the existing “Clear All Viewports” tool.


What Can You do?

Add: `clearSingleViewport` from `ris.ControlViewerFunction`

Viewer Functions [ControlViewerFunction]			
<code>clearSingleViewport</code>	Clear Single Viewport	Clear Single Viewport	Remove

How to Use

1. Right-click on the Clear Viewports tool icon. Select “Clear Single Viewport” from the context menu. This will toggle clear icons on each viewport.

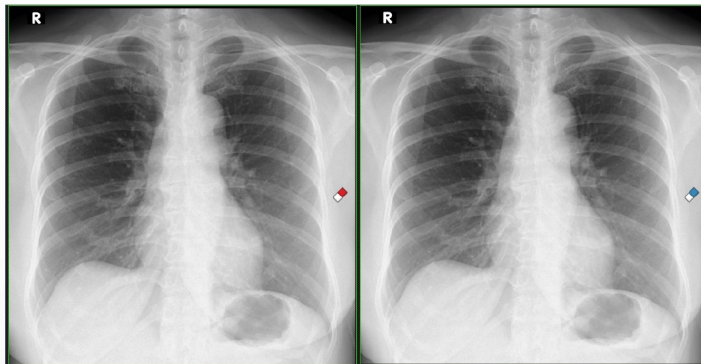

2. Click the Clear icon on the image to clear the selected viewport.

3. To remove the icon from the viewport, right-click Clear Single Viewport again and the icon will disappear

Additional Resources

Click the button below to watch a short video tutorial demonstrating how to use this feature:

Watch