

User Guide

New Features

UPDATED FEATURES

COPY IMAGE FROM VIEWER WITH A BUTTON

User Level

All

Overview

The diagnostic viewer introduces a new tool that enables users to easily copy images into the clipboard which can then be pasted into the chatGPT AI companion using ctrl + v or right click → paste

1. On the diagnostic viewer, look for the copy to clipboard tool on the tools list, add them to the tool bar, and a copy icon should appear on all viewports.



2. Click the icon in order to copy whatever image is in the viewport of the diagnostic viewer to the chatGPT AI companion



Additional Resources

Click the Button below to watch a short video tutorial demonstrating how to use this feature:

[Click Here](#)