

— Changed behavior and Improvement —

Feature Extension

WYSIWYG Google Export

The google export feature is now improved to be able to accept time zone adjustments made on the worklist so users don't need to convert date/time columns to whatever time zone on the spreadsheet. Users should see the same value on the spreadsheet as what they see on the worklist.

How is it used?

After adding some time zone filters or any other filters on any column, click the google export button and name the file. Click the export button and wait for the file to appear on your google drive. Check the data if it matches the worklist.

— Feature Extension —

DV - Single click on thumbnail to load series on viewport

This feature allows users to load a series into the viewport more easily just by single clicking each thumbnail. On the previous versions, users needed to double click the thumbnails in order for them to appear on the viewport.

How it is used

Open the viewer and click any thumbnail. The series should load into the viewport after a single click