

New Feature

Improved Datetime Filters on the Worklist

Overview

Filtering worklist columns by date and time is now more accurate and faster. Several edge cases that previously caused missing or incorrect results have been resolved. The filter interface has been updated to guide you through entering valid date and time values.

Key changes

- 'Equals' and 'Not equal to' now require a full date/time value (YYYY-MM-DD HH:MM:SS).
- 'Begins with' lets you enter a start date (and optional time) to show studies from that point forward.
- 'Ends with' lets you enter an end date (and optional time) to show studies up to that point.
- 'Contains' and 'Does not contain' are disabled for datetime columns (an info alert displays when selected).

Tip: If you use a time zone offset, apply it first before adding any other date/time filters to avoid gaps in your results.

Additional Resources

Click the link below to watch a short video tutorial demonstrating how to use this feature:

[Watch](#)