

Changed behavior and Improvement

Feature Extension

MC Ruler Integration with Presentation State

Overview

The MC Ruler, used for accurate measurements in calibrated units (e.g., mm), can now save its annotations directly into the Presentation State (PR) file.

This enhancement enables radiologists and coordinators to retain ruler annotations across sessions and share them consistently with downstream systems or viewers.

How to Use

When a user measures something on an image using the MC Ruler and saves it, the measurement markings are saved into the PR file. This means the measurements stay with the image and automatically appear again when the image is reopened. Users don't need to re-measure anything.



1. Click On the MC Ruler-Tools



2. Click on Write/Delete Presentation State-Tools



3. Click on Show/Hide Presentation State upon refreshing-Tools