

New Feature


Next Study on Pickable Status

Overview


You can now configure your report study page to automatically advance to the next study on your worklist whenever you set a report to a specific status — for example, automatically moving to the next study after setting one to 'Submitted'.

How to Use

1. On the report study page, locate the 'Next study on' button in the Set Report Status row (shows 'Not set' by default)

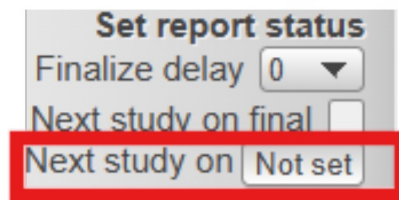

2. Click it to choose the report status group (Default) and status that should trigger the auto-advance.

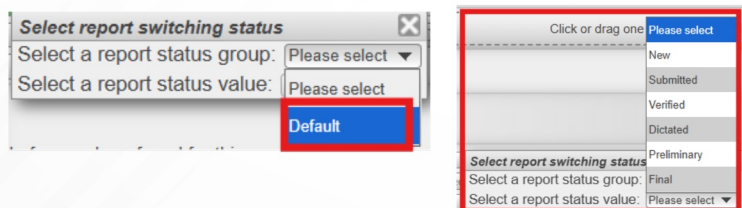

3. No manual save required — your choice is stored per layout.

Note: This feature must be enabled by your PACS Administrator. Different layouts can be configured to trigger on different statuses.

Additional Resources

Click the link below to watch a short video tutorial demonstrating how to use this feature:

Watch