

User Guide

New Features

UPDATED FEATURES

1. Expandable Radpair Column (Final Reporting)

User Level
Radiologists

Overview

The Radpair column in the Final Reporting view is now **resizable**, allowing users to adjust the column width for improved visibility and workflow efficiency.


What's New

The Radpair column can be expanded or reduced in width.

No setup or configuration is required.

How to Use

1. Navigate to the Final Reporting screen.
2. Locate the Radpair column.
3. Drag the resize handle at the bottom-right corner of the column header to adjust the width as needed.

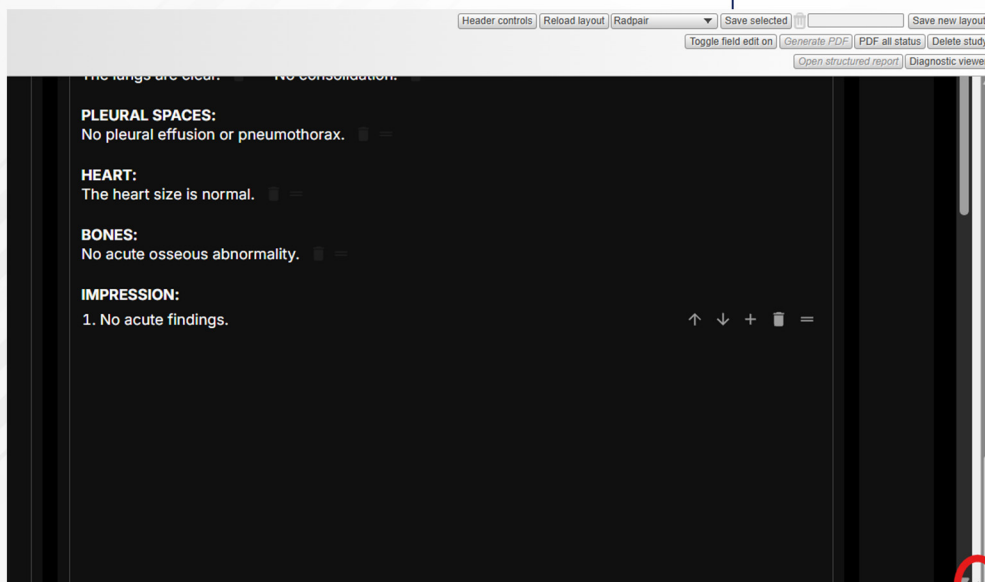

Additional Resources

Click the Button below to watch a short video tutorial demonstrating how to use this feature:

[Click Here](#)