

New Feature

Retract Addendum

Overview

Authorized users can now retract addenda directly from the report page. Retracted addenda are suppressed from standard report output, while the full addendum history is preserved. A retraction reason is always recorded.

What Can You do?

On FAF: Add:


hideRetractedAddendum

RetractAddendum



How to Use

1. On the report study page, click the 'Retract' button next to an addendum entry.
2. Enter a reason for the retraction in the dialog that appears (up to 255 characters), then confirm.
3. To hide retracted addenda from view, use the 'Hide retracted addenda' checkbox on the addendum list row.

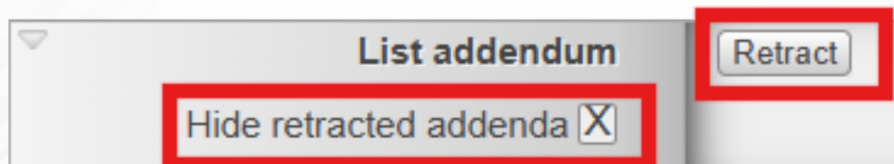

Checking the Audit Trail

All retraction activity is recorded for audit purposes in two ways:

- The reason for retraction is stored against the addendum record and can be retrieved by your administrator if needed.

- Retraction events are also written to the Report Audit log, accessible via Worklist → Report Audit.

Note: Both the Retract button and the Hide Retracted Addenda checkbox must be enabled by your PACS Administrator.

Additional Resources

Click the link below to watch a short video tutorial demonstrating how to use this feature:

[Watch](#)