

Study Size

User Level Administrator

Overview


This column allows users to add another column called "Study Size" to for them to see how big a study is.

Limitations


Data displayed is only in "bytes" so users will have to manually convert it from bytes to either mb or gb.

How to Use

1. Add study Size to the corresponding user's FAF

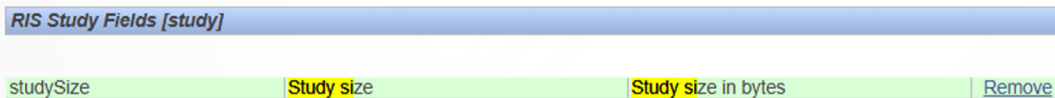

2. To add it to the worklist, select any dropdown buttons from any columns to add another column to the worklist. Look for the Study Size column and check the box. Save the worklist.

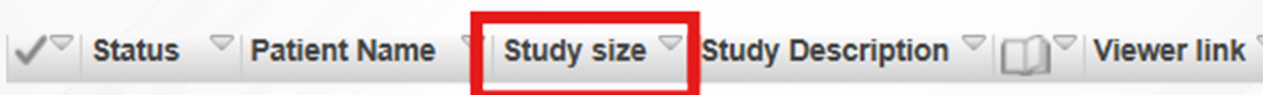

Additional Resources

Click the button below to watch a short video tutorial demonstrating how to use this feature:

[Click Here](#)