

New Feature

Synchronized Pan and Zoom

Overview

The Synchronize Pan and Zoom feature allows users to link multiple viewports (VPTs) into visual groups. Once grouped, any pan or zoom action applied to one VPT will automatically propagate to others in the same group, enhancing multi-panel navigation and comparative analysis.

What Can You do?

On FAF:


Add: syncPanAndZoom

Viewer Functions [ControlViewerFunction]

syncPanAndZoom Synchronize Pan And Zoom Synchronize Pan And Zoom Remove

How to Use

1. Right-click on either the Pan or Zoom tool icon.
2. Select "Sync Pan and Zoom" from the context menu

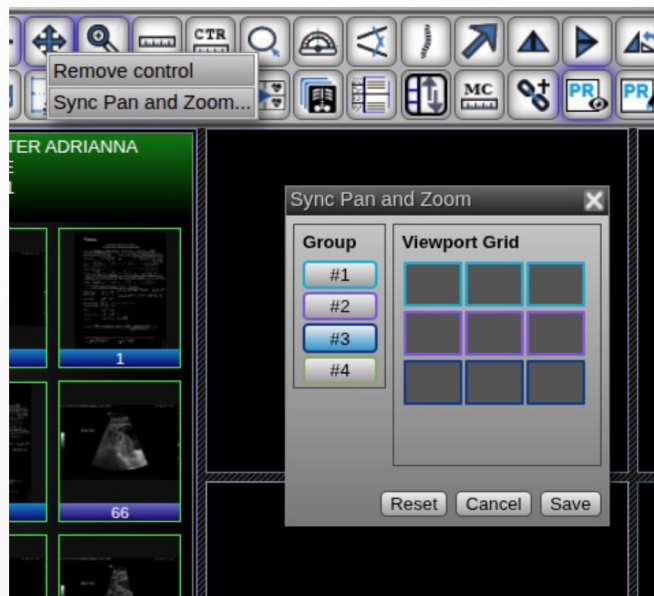

Left: Group Selector (Displays a vertical list of buttons, each representing a group code.) Each button is color-coded for easy identification.

Right: Viewport Grid Layout Mirrors the current layout's row/column arrangement of VPTs. Clicking a grid cell assigns that VPT to the selected group.

3. After grouping is completed, users can pan or zoom all viewports within the same group simultaneously.
4. Make sure to save the layout if you want to keep the changes made.
5. Click the Reset button to clear the group assignments.

Additional Resources

Click the button below to watch a short video tutorial demonstrating how to use this feature:

Watch